**EN Sinomarin® Cold & Flu Relief is a 100% natural product for the relief of nasal symptoms associated with the common cold and flu.** It combines the natural decongestant action (via osmosis\*) of hypertonic sea water (2.3% NaCl) with the well-documented properties -beneficial to the nasal mucosa- of Eucalyptus & Spearmint essential oils and Thyme extract.

The sea water used in **Sinomarin® Cold & Flu Relief** is collected from Cancale Bay at the coast of Brittany in France. The area is well-known for its open seas and strong tidal currents and thus constitutes sea water of outstanding quality, rich in mineral salts and trace elements. The sea water quality is continuously monitored by the French Research Institute for Exploitation of the Sea (IFREMER) under the auspices of the French Government.

The essential oils of **Eucalyptus globulus & Mentha spicata** and **Thymus vulgaris** extract are rich in ingredients (e.g. 1,4-cineole, menthol, carvacrol, etc.) well-known for their benefits on the nasal mucosa and their ability to provide a refreshing sensation. **Sinomarin® Cold & Flu Relief** also contains **Glycerin** offering additional moisturizing properties to the irritated nasal mucosa. All components are 100% natural.

**Sinomarin® Cold & Flu Relief** can be used alone, as a natural alternative to medicated decongestants (local corticosteroids, antihistamines, vasoconstrictors), or in combination with them to help enhance their efficacy.

#### Properties:

- ▶ Relieves nasal congestion and sinus pressure.
- ▶ Fluidifies mucus secretions.
- ▶ Cleanses the nasal cavities removing mucus and mucus-trapped infectious agents (via mechanical action).
- ▶ Provides a refreshing sensation.
- ▶ Moisturizes the nasal mucosa and relieves nasal dryness.
- ▶ Helps promote natural recovery of the nasal mucosa.

#### Sinomarin® Cold & Flu Relief is recommended:

- ▶ For the relief of nasal symptoms associated with the common cold & flu (e.g. nasal congestion, rhinorrhea) and in cases of rhinosinusitis.
- ▶ To eliminate infectious agents that may lead to secondary infections e.g. sinusitis, otitis (via mechanical action).
- ▶ In cases where medicated decongestants are contraindicated (e.g. high blood pressure, heart conditions, diabetes).
- ▶ To help enhance the efficacy of medicated decongestants.

#### Composition:

Sea water (equivalent to 2.3% NaCl), Purified water, Thymus vulgaris extract, Eucalyptus globulus essential oil, Mentha spicata essential oil, Glycerin, Propanediol. Preservative-free. Drug-free. Chemical additives-free. 100% natural.

#### Recommended dosage:

For children over 6 years and adults. Use as per below, or as per physician's instructions. May be used without interruption for up to 30 days.

1 to 2 sprays in each nostril, 2 to 3 times daily.

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**Precautions:** For nasal use only. Do not use same nasal applicator on more than one individuals. Do not use past expiry date. Keep out of reach of children.

**Do not use during pregnancy, breastfeeding or if sensitive to any of the components.**

**Advice:** A mild and temporary stinging sensation or temporary irritation may be experienced at the beginning of use.

**Storage conditions:**

Store below 25 °C. Protect from sunlight.

**Sinomarin®** is a complete line of hypertonic sea water solutions covering the needs of all ages from newborns to adults.

**Sinomarin®** products are CE-marked Medical Devices conforming to the requirements of Directive 93/42 of the European Union.

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## Instructions for use / Gebrauchsanweisung / Kullanım talimatları

- 1 Blow nose to clear out mucus before using **Sinomarin® Cold & Flu Relief**.
- 2 Remove protective cap.
- 3 Hold the spray bottle upright and spray a few times in order to activate the spray pump.
- 4 Bend neck gently forward over a wash basin and tilt head to the right.
- 5 Gently insert applicator into the left nostril in parallel with the nasal septum, and press firmly.
- 6 Turn head to opposite side and repeat application in other nostril.
- 7 Remove spray from nostril, return to upright position while allowing solution to work for a few seconds and then blow the nose.
- 8 Repeat procedure if necessary.
- 9 Wash applicator well with warm water and wipe dry after each use.

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